

IMPORTANT NOTICE REGARDING SOFTWARE VERSION COMPATIBILITY

WARNING:

Do not use DTR-300 Training Software Cards with upgraded DDU-100 Rescue AEDs.

If a DTR-300 Training Software card and a red Training Battery Pack are used for training in a DDU-100 AED operating with software version 2.001 or greater, the unit will not speak while in training mode (although the unit will operate normally with a rescue battery installed).

NOTE:

This warning only applies to the use of red Training Battery Packs with DDU-100 Rescue AEDs for training.

You are not affected by this warning if:

- You do not use a red Training Battery Pack with your DDU-100 Rescue AED for training.
- You use a Standalone AED Trainer for training.

ACTION TO TAKE:

If you do use a red Training Battery Pack System with your DDU-100 Rescue AED for training, check the label of your Training Software Card:

- If your card is a DTR-301 card (pictured here), you are not affected by this warning and may use your upgraded DDU-100 AED for training.
- If your card is a DTR-300 card (pictured here), do not use your DDU-100 Rescue AED for training with this DTR-300 card. Please contact Defibtech for a free replacement card at techsupport@defibtech.com, 1-877-453-4507, or 1-203-453-4507.

