

## How to Determine Your Defibtech AED's Software Version

- ① **Verify that a Data Card is not installed** in the AED.
  - Remove the Battery Pack from the AED by pressing the orange release button (Figure 1).
  - Remove the Data Card (if installed) from the card slot (Figure 2).
  - Replace the Battery Pack. A brief Battery Pack Self Test will begin.
  - Allow the test to complete. The AED will turn off.
- ② **Turn the AED ON** by pressing the ON/OFF button once.
- ③ After the unit begins speaking, **press and hold the ON/OFF button** for approximately 5 seconds. You will hear “Data Card Missing” and:
  - If the AED has software version 1.201:** The AED will turn off without announcing a software version. *You will know that the software version is 1.201 if the AED does not announce any version number before powering off.*
  - If the AED has software version 1.202 or greater:** The AED will announce the software version before turning off. *You will hear “Version, #,#,#,#” to indicate that the software version is #.###.*
- ④ If a Data Card was installed, replace Data Card.

If you have any questions or concerns about the process, contact your distributor or Defibtech Customer Service at (866) 333-4241.



Figure 1 – Press the orange release button to eject the battery pack from the AED.



Figure 2 – Verify there is no data card installed in the card slot. If there is a data card installed, remove the data card by pushing the card in and releasing. The card will partially eject and can then be removed.