

REF RBP-1000

Rechargeable Battery Pack for the Defibtech
RMU-1000 Automated Chest Compression System

RMU-1000 User Manual Addendum*



RBP-1000 Rechargeable Battery Packs that carry this symbol as shown in *Figure 1* at right are only compatible with RMC-1000 Compression Modules running system software version 1.109 or greater.



WARNING

RBP-1000 Rechargeable Battery Packs with this marking will not function with a RMC-1000 Compression Module running system software version 1.105 or lower. When the Module is powered on with a Battery Pack with the marking, the Module will indicate a battery error and the RMU-1000 ACC cannot deliver therapy, even if the external power supply is plugged in to the Compression Module.


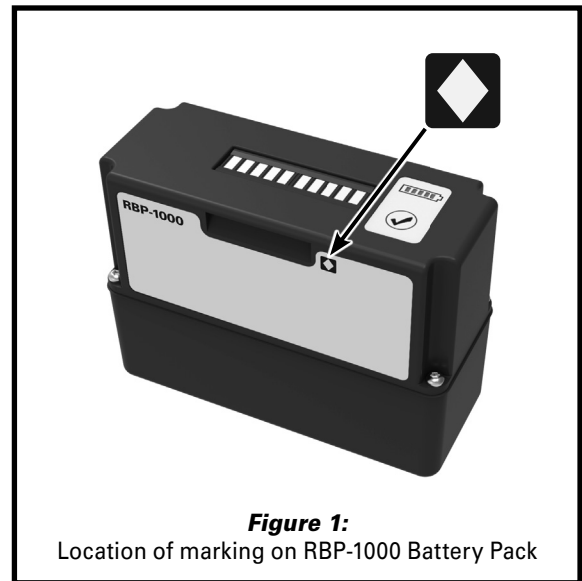


Figure 1:
Location of marking on RBP-1000 Battery Pack

To check what system software version number that any RMC-1000 Compression Module is running:

1. Go to www.defibtech.com/eventconnect to download *ACC Event Connect* software, making sure that your PC meets the minimum system requirements listed. Launch the installer and follow the on-screen instructions until installation is complete.
2. Connect the RMC-1000 Compression Module to the host PC via a USB cable (USB-A to Mini-B). The Module's USB (Mini-B) port is located on the underside of the device, as shown at right. Power on the Module by pressing and holding its power button for at least one second.
3. Launch *ACC Event Connect* software by double-clicking the *ACC Event Connect* shortcut icon (shown at right)

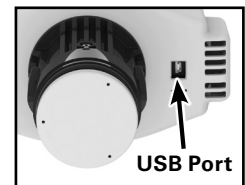

USB Port

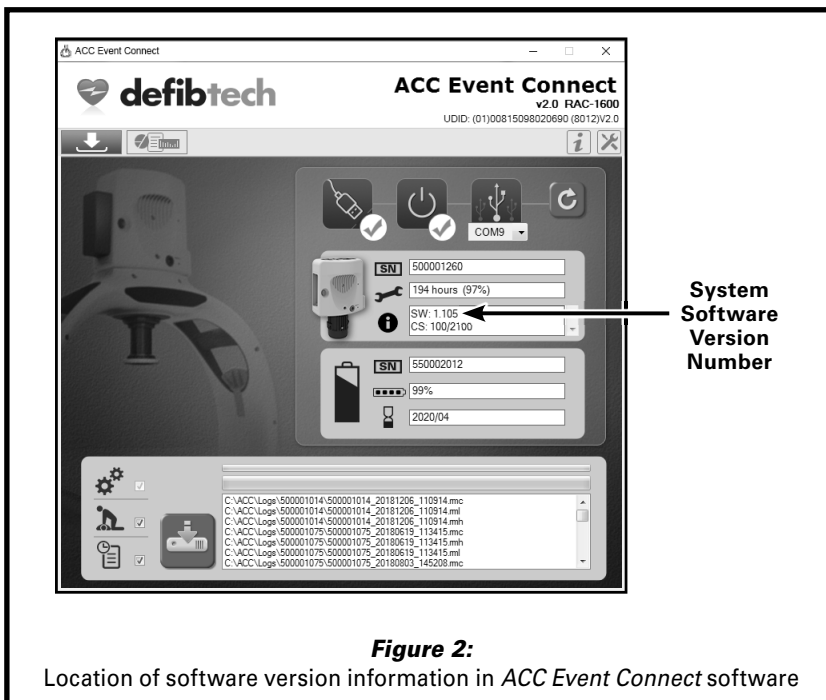


Figure 2:

Location of software version information in *ACC Event Connect* software

4. When *ACC Event Connect* is launched, the screen shown in *Figure 2* at left will appear. If the Compression Module is powered on and is properly connected to the host PC via USB, the system software version number will be displayed in the indicated location.

5. If the system software version displayed is **"SW: 1.109" or greater**, the RMC-1000 Compression Module is fully-compatible with all RBP-1000 Rechargeable Battery Packs. If the system software version displayed is **"SW: 1.105" or lower**, the RMC-1000 Compression Module will not be compatible with RBP-1000 Rechargeable Battery Packs that carry the marking shown in *Figure 1* above. **Contact your authorized distributor or Defibtech using the contact information shown below.**

**Defibtech recommends that this addendum be retained with the RMU-1000 ACC User Manual.*



Defibtech, L.L.C.

741 Boston Post Road, Suite 201
Guilford, CT 06437 • USA

Web: www.defibtech.com

Tel: 1-(866)-333-4241 (toll-free within North America) • 1-(203) 453-4507

Email: service@defibtech.com (service and repair); sales@defibtech.com (sales)

*Refer to the RMU-1000 ACC User Manual for device set-up, instructions, and troubleshooting.
For additional information about ACC Event Connect software, refer to the RAC-U1600
ACC Event Connect Software Instructions available at www.defibtech.com/eventconnect*

ARM-GD008-100-EN

ELECTRONIC DISTRIBUTION

Issued: 2019-03-25